**Bedienungsanleitung  
Instruction manual  
Manual de instrucciones**

**Cellcounter**

Elektronisches Blutbildzählgerät mit Digitalanzeige  
Electronic blood counter with digital display  
Aparato electrónico para recuento de células con pantalla digital

## Inhalt

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## Content

1. Scope of delivery	15
2. General information	15
2.1 Unpacking the instrument	15
2.2 Safety instructions	15, 16
3. Intended use and description	16
3.1 Intended use	16
3.2 Description	16, 17
4. Installation	18
5. Setup	18
5.1 Level 1: Select and confirm the program	18
5.2 Level 2: Select and confirm stepwidth and countstop	18, 19
5.3 Level 3: Free programming of the counting keys	19
6. Counting	19
6.1 Program Leuko	20
6.2 Program Myelo	20
6.3 Program Reti	20
6.4 Program Free	20
7. Evaluation	20, 21
8. Saving the counts and program parameters	21
8.1 PC communication	21
9. Cleaning and maintenance	22
10. Dismantling and disposal	22
11. Transport and storage	22
11.1 Packing / transport	22
11.2 Storage	23
12. Warranty and Liability	23
13. Technical data	24

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## 1. Scope of delivery

- 1 Cellcounter cat. No. 7170006
- 1 External USB power supply
- 1 Instruction manual

## 2. General information

The instrument is designed in accordance with safety class II. It is built and tested in accordance with DIN EN 60335-1; VDE 0700-1:2012-10:2012-10, safety regulations for electrical instruments. It left our warehouse working perfectly well and safely. To maintain the proper safety and operational function of the instrument, the user must respect the instructions and safety guidelines of this manual.

The device is supplied with a USB power adapter 5VDC / 300 mA as standard and can be operated with an external USB power bank.

### 2.1 Unpacking the instrument

Unpack the instrument carefully and check to see that it is not damaged. It is important that any damage incurred during transport, such as indented sides of the carton, chemical impairment of the carton, water stains or other physical impacts which might have damaged the content, be recognized at the time of unpacking. Notify your carrier or forwarding agent immediately in case of such damage. You may unpack the carton carefully if there is no visible damage of the packing material.

### 2.2 Safety instructions

**Note:**

If the instrument is not damaged and all parts are complete, you may start to operate the device after reading the instruction manual.

**Important note:**

Read this instruction manual carefully and make sure everything is understood before operating the instrument. Should there be any further questions after reading the manual concerning the setup, operation or warranty, please contact:

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**Attention:**

Please comply with all safety and accident-prevention regulations applicable to laboratory work.

**Caution:**

When connecting the instrument to an AC power outlet, ensure that your local supply voltage corresponds to the specifications indicated on the instrument.

**Caution:**

The On/Off switch does not disconnect the instrument from the power source. Remove the plug from the AC power outlet to disconnect the instrument from the mains supply entirely.

**Attention:**

Do not open the instrument! Repairs are only to be carried out by trained service technicians. Any work on the electronics of the unit should only be done by knowledgeable and trained personnel. In the case of unauthorized operation as well as negligent or wilful damage, the warranty expires.

### 3. Intended use and description

#### 3.1 Intended use

The Cellcounter is an electronic device for registering counts, storing and statistical evaluation of results. It is specially designed for counting blood cells, but is also suitable for counting other particles.

#### 3.2 Description

The Cellcounter is an electronic count instrument with 4 programs, 15 counting keys and 5 operation keys.

Programs: LEUCO, MYELO/RBC, RETI, FREE


Counting steps: 50-1000 in steps of 50

Counting keys: At Ly, My Bl, Met My, Myelo, Normo, Lym, Seg, Mono, Plas, Tox Gr., Stab, Eos, Baso, Div/Reti, Div/Ery


**Layout Keyboard:**

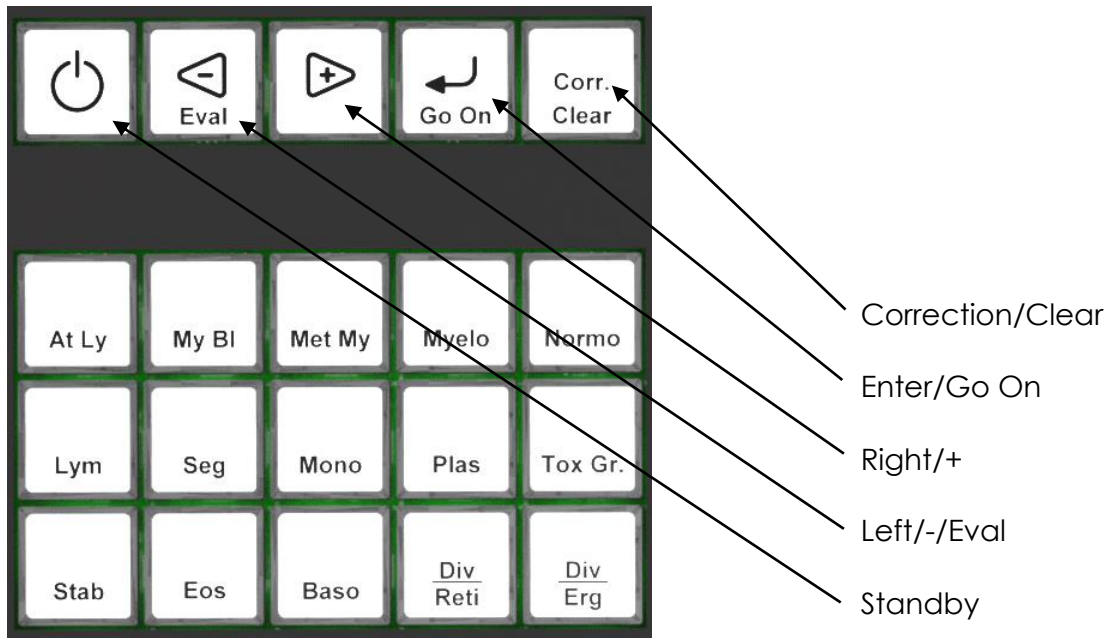**Display:**
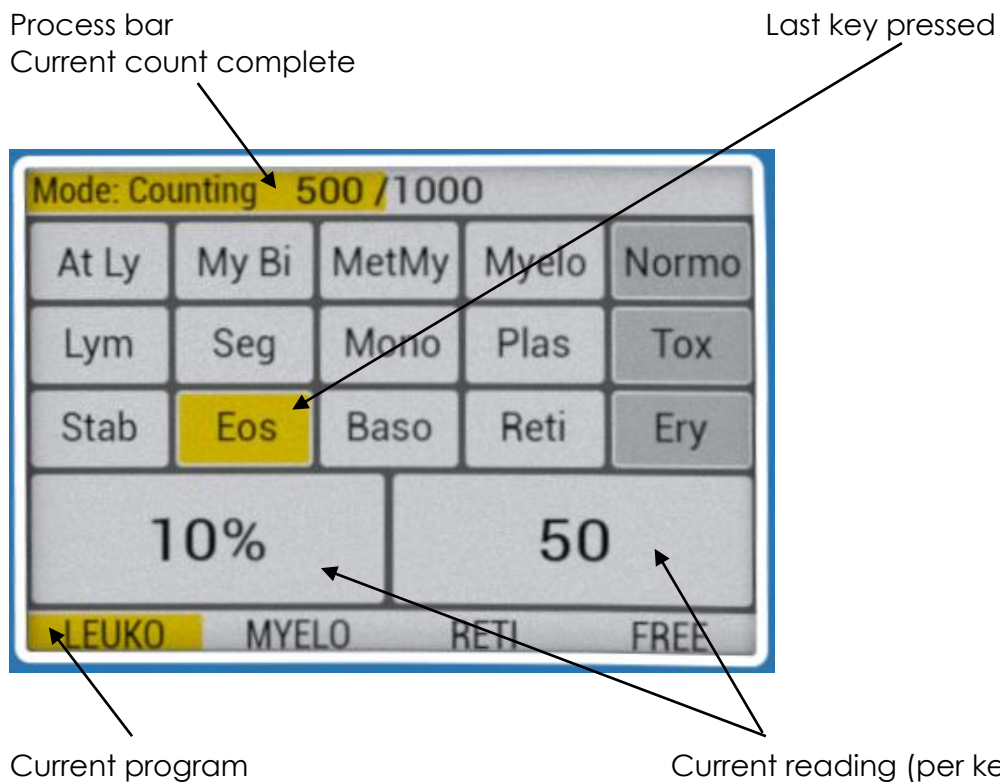
## 4. Installation

After connecting to a supply voltage, the device is in sleep mode and must be started. This process is accompanied by a short audio signal, which provides additional feedback.

If the device is in standby, it is started via the standby button. After starting the device, the last program used is loaded. The setup can be skipped by pressing the "Enter / Go on" button twice (three times for the "Free" program).

The device is switched off correctly by holding the standby button.

## 5. Setup

The settings for the counting process are made in the setup phase of the program. In this phase, depending on the counting program, either the stepwidth or the countstop and exclusive counting keys are set.

### 5.1 Level 1: Select and confirm the program

The program is selected using the „Right/+“ or „Left/-/Eval“ buttons.

Now the last used program setting is loaded. By pressing any counting key the rest of the setup phase can be skipped and the last count is displayed (option only in level 1).


Confirm the selection by pressing the key „Enter/Go On“. The display now changes to level 2 of the setup.

### 5.2 Level 2: Select and confirm stepwidth and countstop

#### a. Program „Reti“

When selecting the program Reti in level 1, the step size for the key "Div/Ery" is to be selected in level 2. By pressing the "Right/+" key, the next higher value from the series 1, 5, 10, 20, 30, 40, 50 is selected. As soon as the value reaches 50, further inputs on this key will be ignored.

The next lower value is selected with the key "Left/-/Eval". Any further entry for this key is ignored when the value reaches 1.

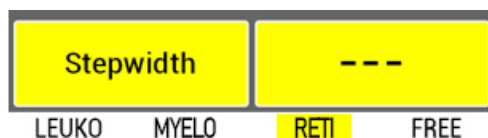Confirm your selection by pressing the key „Enter/Go On“.

**b. Other programs**

When using the other programs, the countstop has to be selected.

By pressing and holding the "Right / +" or "Left / - / Eval" keys, the countstop will increment or decrement in steps of ten.

Confirm your selection by pressing the key „Enter/Go On“. The display will now change to level 3.

**5.3 Level 3: Free programming of the counting keys**

Using the program „Free“ you have to select the exclusive keys. By pressing a counting key which is currently not considered exclusive, it will be marked as exclusive. Pressing an exclusive key will mark it as non-exclusive. Confirm your selection by pressing the key “Enter/Go On” and the display changes to the counting phase.

For all other programs, the exclusive keys are set automatically.

**6. Counting**

Countmode 500 / 1000				
At Ly	My BI	MetMy	Myelo	Normo
Lym	Seg	Mono	Plas	Tox
Stab	Eos	Baso	Reti	Ery
50		10%		
LEUKO	MYELO	RETI	FREE	

After the setup a keyboard is displayed on the TFT color display. It shows which keys are evaluated exclusively and which key was last pressed. If an exclusive key was pressed, it is highlighted in the same way like keys which are non-exclusive.

At the top of the color display the relative reading is shown (if the key is a non-exclusive key) as well as the absolute count of the last key pressed. The exclusive status of a key can thus be seen because no relative value is displayed.

For every counted cell the corresponding key has to be pressed once. By pressing the key “Corr./Clear” the last count can be corrected.

The evaluation mode can be called up during the counting process by pressing the key "Left/-/Eval". If the countstop is reached during a count, the device automatically switches to the evaluation mode.

### 6.1 Program Leuko

- The counting keys of the last column count exclusively: Normo, Tox Gr. und Div/Ery
- The countstop is set in level 2. Briefly pressing the "Left / - / Eval" or "Right / +" keys decrements or increments by 100. By holding the two keys, the countstop is decremented or incremented in steps of 10.

### 6.2 Program Myelo

- No keys are evaluated exclusively.
- The countstop is set in level 2. Briefly pressing the "Left / - / Eval" or "Right / +" buttons decrements or increments by 100. By holding the two buttons, the countstop is decremented or incremented in steps of 10.

### 6.3 Program Reti

- All buttons except "Div / Ery" are counted exclusively.
- The countstop is set to 1000. In level 2 of the setup a step width for "Div / Ery" is defined (1, 5, 10, 20, 30, 40, 50).

### 6.4 Program Free

- In Level 3, the user can freely select the keys that count exclusively.
- The countstop is set in level 2. Briefly pressing the "Left / - / Eval" or "Right / +" buttons decrements or increments by 100. By holding the two buttons, the countstop is decremented or incremented in steps of 10.

## 7. Evaluation

Evaluation 500 / 1000				
At Ly	My Bl	MetMy	Myelo	Normo
Lym	Seg	Mono	Plas	Tox
Stab	Eos	Baso	Reti	Ery
50		10%		
LEUKO	MYELO	RETI	FREE	

The results can be read on the color display. In contrary to the counting process, the absolute and relative counts are highlighted. When pressing the counting keys now, counting is no longer triggered. The display of the counts is switched according to the respective key pressed.

The evaluation can be terminated by pressing the key „Enter/Go On“:

- If the countstop has not yet been reached while counting, the screen changes back to the counting process.
- If the countstop has already been reached, it is incremented by 100 (unless it did not already reach the value 1000). The display now changes to the counting mode.
- If the countstop had already reached the value of 1000, the key „Enter/Go On“ is ignored. The menu can now only be exited when resetting the device by holding the "Corr./Clear" key.

## 8. Saving the counts and program parameters

When switched off correctly (holding the standby button), the device saves the last program set and the last created counting data.

If an incomplete count has been saved, the device starts directly in the counting mode with the counting data of the incomplete count when the device is switched on again (by pressing the standby key). You can continue counting now. When the saved count has been completed, it can be called up in level 1 by pressing any counting key.

### 8.1 PC communication

The device transfers the generated counting data to the Cellcounter PC-Manager (software available for download from our website). To do this, the device must be connected to a computer using a suitable USB cable. In addition, when a key is pressed, the associated key code is sent. The codes and associated keys are shown in the table below.

Name	Standby	Left/-/Eval	Right/+	Enter/Go On	Corr./Clear
Code	1	3	4	5	6
Designation	Counting key 1	Counting key 4	Counting key 7	Counting key 10	Counting key 13
Key	66	72	78	84	90
Designation	Counting key 2	Counting key 5	Counting key 8	Counting key 11	Counting key 14
Key	68	74	80	86	92
Designation	Counting key 3	Counting key 6	Counting key 9	Counting key 12	Counting key 15
Key	70	76	82	88	94

## 9. Cleaning and maintenance

The case is made of acid-resistant material and easy to clean. Surface and operating elements may be cleaned with a mild dishwashing detergent (water and a standard dishwashing detergent) and a soft, lint-free **moist** cloth. Under no circumstances the surface should be cleaned with acids, chlorinated solvents or saline solutions.

Do not rub or scratch to avoid damages of the inscriptions and the surface. If you use any other cleaning method, please make sure that the intended method does not cause any damage to the instrument.

### Attention:

Carefully prevent that no liquid penetrates into the spaces between the keys. Do not rinse the instrument.

## 10. Dismantling and disposal

1. Switch off the instrument by means of the rocker switch.
2. Disconnect the instrument from the mains.
3. Now you can remove the instrument from the working area.

Damaged and/or disposed electric or electronic equipment has to be disposed of at the designated recycling depot.

Also the packing material should be disposed of environment-friendly (material separation).

## 11. Transport and storage

### 11.1 Packing/Transport

Please use, if possible, the original packaging. Protect the instrument by means of bubble wrap against mechanical shocks from the outside. In case you do not use the original packaging, please mark the box with the following notes:

- Glass symbol (Fragile! Handle with care!)
- Umbrella (Keep dry!)
- Content (List of content)

## 11.2 Storage

For proper storage please observe the following conditions of surrounding:

max. ambient temperature: 5-40 °C  
max. ambient humidity: 80 %

## 12. Warranty and Liability:

### Warranty:

Paul Marienfeld GmbH & Co. KG provides a guarantee for a period of 24 months from date of delivery for the instrument purchased by you. This warranty applies only to defects in material or workmanship. There is no warranty for defects or malfunctions caused by neglect, improper use or improper service.

In case of defects in material or workmanship, the instrument will be repaired or replaced. In case of replacement, the warranty for the new unit will persist for the remaining period of primary warranty period. Charges for returning the instrument must be borne by the customer.

Any claims regarding guarantee and liability are limited to professional and proper use of the instrument and observance of the general safety instructions and this instruction manual.

### Liability:

Paul Marienfeld GmbH & Co. KG is not liable for any material or personal damages caused by improper use and does not accept liability for consequential damages. The warranty expires in case of removal or alteration of the identification label and/or the serial number of the instrument.

### Attention:

In case of malfunction do not try to carry out any repair works. The instrument does not contain any part which may be serviced or maintained by the user. In case of any attempt by the user to repair the unit, the warranty expires!

Do not open the instrument. Any work on the electronics of the unit should be done by knowledgeable and trained personnel only!

### 13. Technical data

Number counting keys:	15
Number operation keys:	5
Display:	TFT color 3,5"
Keyboard:	Tactile and acoustic feedback
Dimensions (LxWxH):	Instrument: 235 x 130 x 35 mm Packed: 300 x 230 x 90 mm
Weight:	0,9 kg gross
Interface:	USB Mini
Mains operation:	USB Power adapter 5VDC / 300mA
Battery operation:	External USB power bank (not included)

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